fisianal

Istruzioni d'uso

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Sonorous biofeedback
for home rehabilitation of
the posterior perineal muscles

WHAT IS FISIANAL?

FISIANAL is a multi-purpose medical device made of an inert plastic material and used to improve various primary or secondary anal pathological conditions; it is used to treat post-operative rehabilitation of the anus and rectum, to improve the sphincter tone, to promote healing and significantly reduce postoperative pain and the risk of scarring stenosis. **FISIANAL** is also a practical home biofeedback device which stimulates the contraction of the muscles of the anal sphincter.

THERAPEUTIC INDICATIONS

- Anorectal postoperative rehabilitation after haemorrhoidectomy, lateral sphincterotomy and excision of perianal fistula operations. If there are stitches begin therapy with **FISIANAL** about 10 days after surgery.
- Prevention of postoperative stenosis after surgery for haemorrhoids, anal fissures and perianal fistula.
- Hypertonia or sphincter spasm supported by haemorrhoids, fissures, essential proctalgia with possible secondary constipation.
- Initial stenosis of the anal orifice of scar or idiopathic origin.
- Rehabilitation of the pelvic floor.
- Anal hypotonia (increased muscle tone).

HOW TO USE IT

The design of **FISIANAL** is ergonomic and adaptable to the anatomy of the anus and anal canal. It consists of a flexible introducer with a 17mm diameter covered by a soft protective sheath. An internal membrane registers the pressure of the anal muscles once contracted. When the exercise is performed properly two indicators are activated, one visual and one sonorous: the control unit turns on a blue LED and at the same time you hear a beep. The selector dial at the base of the introducer has a graduated scale from 1 to 10 that increases the external rigidity of the instrument, a characteristic that allows it to adapt well to different kinds of treatment. For proper use, the selected value must be set at the adjustment index. Before use, thoroughly lubricate the introducer with water-soluble gel (LuJ-Sapi Med) or with a cream for anorectal diseases (Camocal-Sap Med). After use, release the plug from the introducer with a counterclockwise rotation.

TREATMENT METHOD

It is recommended, in the case of a sphincter spasm, to precede the treatment with a warm-tohot hip bath to relax the muscles of the anal sphincter. On the dial, select the value recommended by the medical specialist at the beginning of the therapy. Carefully lubricate the introducer. While lying down on the side (fig.3), either before or after defecation (never on the bidet or sitting), gently and slowly insert the introducer into the anal orifice as far as the base. Press the power button (On) on the control unit, the green LED lights up. **FISIANAL** should be kept in place by performing exercises of contraction and relaxation of the anal sphincter: perform at least two sets of 20 with a short pause between the first and the second. **If the sphincter contraction exercises are performed correctly, the device beeps and the blue LED on the control unit lights up.** If the signals do not function, decrease the resistance of the introducer by selecting a lower value on the scale or ask your doctor if the exercise is performed correctly. When the treatment is over, slowly extract the introducer by grasping the base. Do not pull **Fisianal** by the cable. Do not get up and do not walk when the dilator is inserted.

FREQUENCY

First week two applications per day.
Second week one application per day.
Third and fourth week one application every other day.
If pain persists, continue therapy as directed by the prescribing physician.

CLEANING THE FISIANAL

- Release the electrical part of the device by turning the plug a quarter turn counterclockwise.
- Wash only the introducer with neutral soap under warm running water after each use.
- Do not immerse.
- Dry thoroughly with a soft cloth and store in a dry and protected place.

DO NOT USE IN CASE OF:

- Recent surgery on rectal anastomosis with low or ultra-low colorectal anastomosis or colo-anal anastomosis or total proctocolectomy surgery with ileo-pouch.
- Fistulizing perianal Crohn's disease.
- Ulcerative rectocolitis undergoing severe activity.
- Rectal neoplasms or polypoid lesions revealed by the rectal ampulla.
- Tightened scarred anal stenosis.
- Pediatric and neonatal age.

WARNINGS

A proctological examination should be performed by a qualified practitioner to diagnose the origin of the symptoms.
Keep out of reach of children.
Do not swallow.
Do not freeze.
Do not heat.
Do not immerse any part of the device in water.
Do not attempt to open or disassemble the device.
Do not dispose of in the environment after use.
The use is strictly personal.
Any colour changes do not compromise the correct operation of the device.
If the green LED does not light up, replace the battery.
Dispose of in accordance with applicable law.

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