I DILATAN® - DILATATORE ANALE CRIO-TERMICO
3 MISURE: Ø 20 mm - 23 mm - 27 mm

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GB **DILATAN® - CRYOTHERMAL DILATOR
3 SIZES: Ø 20 mm - 23 mm - 27 mm**

The DILATAN® cryothermal dilator is manufactured from a white, inert plastic material and supplied in a single-patient box of three different sizes. Each sealed dilator contains a thermal jelly which, when heated or cooled, maintains the dilator's temperature during use. Thermotherapy (warm) or cryotherapy (cold) dilators may be used effectively to treat differing pathologies. For the following conditions the dilator should be used warm, up to but not exceeding 50°C:

- Stenosis (contraction) of the anus, due to cicatricial or idiopathic origi
- Hypertonia or spasm due to, and sustained by, haemorrhoids, proctalgia or fistulas
- Prevention of post-operative stenosis due to haemorrhoids or fistulas

For the following conditions, the dilator should be cooled to not below 0°C: • Anal bleeding In other pathologies, the temperature of the dilator is not important. The small and medium-sized dilators are to assist in progressive distension in the presence of various anal stenoses. The large size is always indicated for spasm.

DIRECTIONS FOR USE

- For THERMOTHERAPY, the dilator should be immersed in warm water, ideally at 50°C, for 10-15 minutes. For CRYOTHERAPY, the dilator should be immersed in ice water for 15 minutes in the fridge. The DILATAN® works well regardless of whether the lower bowel is full or empty. In case of spasm, sitting in a warm water bidet prior to insertion will help relax the sphincter muscles and assist the therapy. Prior to insertion, lubricate the dilator with an ointment. Sitting in the bidet or lying on the bed, insert the dilator slowly up to its base. Keep it in position for two to three minutes, and then pull it slowly out. It is advisable not to stand or walk with the dilator in position.
- Following haemorrhoidectomy, dilations should be performed once or twice daily for the first month, reducing to three times a week for the second and third months. This will greatly relieve post-operative discomfort.
 - In cases of anal stenosis, twice-daily use of the dilator is advisable. Treatment should be continued for several months.
 - For other conditions, DILATAN® should be used twice a day whilst symptoms persist, and the treatment continued for at least a month after the symptoms have ceased.

CLEANING DILATOR

Following use, the dilators should be washed in warm, soapy water, thoroughly rinsed and returned to the box when clean and dry.

Warnings

- It is recommended a proctological examination to avoid a different origin of symptoms.
- Do not boil in liquid of any type.
- Do not freeze.
- Do not heat in microwave.
- Do not dispose of as household waste.
- Dispose of as hospital waste in accordance with local regulations.
- **DILATAN® is a single-patient product.**

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