
Dilatatore DILASTOM® di calibro minore
Smaller calibre DILASTOM® dilators
Dilatador DILASTOM® de calibre inferior
Menší dilatátor DILASTOM®
Menší dilatátor DILASTOM®
Najmanjši dilatator DILASTOM®
Dilatateur DILASTOM® plus petit
Dilatador DILASTOM® menor
Kleinerer DILASTOM® dilatator
Kleinste maat DILASTOM®

Dilatatore DILASTOM® di calibro superiore
Larger calibre DILASTOM® dilators
Dilatador DILASTOM® de calibre superior
Větší dilatátor DILASTOM®
Větší dilatátor DILASTOM®
Večší dilatátor DILASTOM®
Dilatateur DILASTOM® plus grand
Dilatador DILASTOM® maior
Größerer DILASTOM® dilatator
Grootste maat DILASTOM®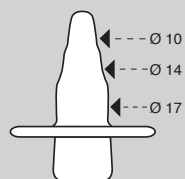

Modalità d'uso
Direction for use
Modo de empleo
Způsob použití
Sposob použitia
Navodila za uporabo
Mode d'emploi
Modo de uso
Gebrauchsweise
Gebruiksaanwijzing

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STOMA DILATOR

INSTRUCTIONS FOR USE

Among the complications that can arise with a colostomy, stenosis, or narrowing of the stoma, occurs in about 8-10% of cases, according to data on experience throughout the world.

The correct definition of STENOSIS must take the size of the stoma, the elasticity of the parietal tract of the intestinal wall, and the capacity of the surrounding tissue to stretch into account.

Stenosis may be EARLY or LATE, depending on the interval between the surgery and their appearance, and CUTANEOUS or FASCIAL, depending on whether the narrowing is at the skin level or in the muscle aponeurosis. When narrowing or stenosis of a colostomy occur, faeces elimination and irrigation procedures can become very difficult. The aim of dilating treatment is to block the progressive narrowing of the external orifice of the colostomy. The passage of faeces of hard consistency through the stenotic lumen can cause sharp pain. In addition, stenosis makes irrigation very problematic, since introduction of the irrigation cone and water is difficult, and sometimes requires adjustments which can be risky.

USE

There are two dilators, conical in shape to facilitate their introduction into the stoma. The smaller has perfectly smooth walls, and allows dilation up to 15 mm; the larger has stepped walls of increasing diameter to produce up to 17 mm gradual and lasting dilation. Start dilating with the smaller calibre **DILASTOM[®]**. Lubricate the device thoroughly with Sapi Med Lu J (lubricating jelly) or petroleum jelly. Turn the device gently with steadily increasing pressure until the **DILASTOM[®]** has fully penetrated. Maintain the stoma dilator in position for 10-15 minutes as prescribed by your doctor and repeat the operation at least twice a day. The dilating treatment must be performed regularly, every day, for cycles of 1-2 months, as advised by your specialist. The next phase, always as advised and recommended by your specialist, is to proceed to use the higher calibre device with the same application times. The conical shape and the mirror-smooth walls of **DILASTOM[®]** mean that, once introduced, it can remain in place by simple pressure of the fingers on the base of the dilator. Small and brief episodes of bleeding are no cause for alarm. **DILASTOM[®]** stoma dilator is supplied in a practical case containing both sizes. Prolonged use of the device does not cause any unwanted side effects, and, on the contrary, is capable of reducing or eliminating the need for surgery to correct the stenosis.

CLEANING THE DILASTOM[®]

The **DILASTOM[®]** should be cleaned every time it has been used with soap and water or disinfectant and water, dried, and replaced in its case.

WARNINGS

Keep out of children’s reach. Do not swallow. Do not boil. Do not boil. Latex free. Dispose of as hospital waste in accordance with local regulations. ***The device MUST be used by a single individual only.***

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